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PLEASURE LATEX PRODUCTS SON BHD

LOT 5322, 15TH MILES, JALAN PADANG GAJAH,

45800 JERAM, SELANGOR DARUL EHSAN, MALAYSIA

EC REP

OBELIS S.A.

BOULEVARD GÉNÉRAL WAHIS 53,

1030 BRUSSELS, BELGIUM

TEL:+(32)2.732.59.54

E-MAIL: MAIL@OBELIS.NET

MADE IN MALAYSIA

Rev.2023/03

Use by date / Utilizar por fecha / Utiliser par date / Verfallsdatum / Usare entro la data / Gebruiken bij datum / Usar por data / Spotřebujte do data / Użyj do daty / Använd senast datum / Sidste anvendelsesdato / Best før dato

Date of manufacture / Fecha de fabricación / Date de fabrication / Herstellungsdatum / Data di produzione / Productiedatum / Data de fabrico / Datum výroby / Data produkcji / Tillverkningsdatum / Fremstillingsdato / Produktionsdato

Keep dry / Mantener seco / Tenir au sec / Trocken halten / Mantenere all'asciutto / Droog bewaren / Manter seco / Skladujte v suchu / Przechowywać w suchym miejscu / Förvaras torr / Opbevares tørt / Opbevares tørt

Keep out of Sun / Manténgase alejado del sol / Tenir à l'abri du soleil / Halten Sie sich von der Sonne fern / Tenere lontano dal sole / Blijf uit de zon / Mantenha-se fora do Sol / Chraňte se před sluncem / Chronić przed słońcem / Håll dig borta från solen / Hold dig ude af solen / Hold deg unna solen

Do not re-use / No reutilizar / Ne pas réutiliser / Nicht wiederverwenden / Non riutilizzare / Niet hergebruiken / Não reutilize / Nepoužívejte znovu / Nie używać ponownie / Återanvänd inte / Må ikke genbruges / Ikke gjenbruk

Batch code / Código de lote / Code du lot / Batch-Code / Codice lotto / Batchcode / Código de lote / Kód šarže / Kod partii / Batchkod / Batch kode / Batchkode

Contains natural rubber latex / Contiene látex de caucho natural / Contient du latex de caoutchouc naturel / Enthält Naturkautschuklatex / Contiene lattice di gomma naturale / Bevat natuurrubberlatex / Contém látex de borracha natural / Obsahuje přírodní kaučukový latex / Zawiera kauczuk naturalny / Innehåller naturgummilátex / Inneholder naturgummilátex / Inneholder naturlig gummilátex

Do not use if package is damaged / No lo use si el paquete está dañado / Ne pas utiliser si l'emballage est endommagé / Nicht verwenden, wenn die Verpackung beschädigt ist / Non utilizzare se la confezione è danneggiata / Niet gebruiken als de verpakking beschadigd is / Não use se o pacote estiver danificado / Nepoužívejte, pokud je obal poškozen / Nie używać, jeśli opakowanie jest uszkodzone / Använd inte om förpackningen är skadad / Må ikke bruges, hvis pakken er beskadiget / Ikke bruk hvis pakken er skadet

Consult instructions for use / Consultar instrucciones de uso / Consulter les instructions d'utilisation / Gebrauchsanweisung beachten / Consulta le istruzioni per l'uso / Raadpleeg de gebruiksaanwijzing / Consulte as instruções de uso / Přečtěte si návod k použití / Zapoznaj się z instrukcją użytkowania / Se bruksanvisningen / Se brugsanvisningen / Se bruksanvisning

Manufacturer / Fabricante / Fabricant / Hersteller / Produttore / Fabrikant / Fabricante / Výrobce / Producent / Tillverkare / Producent / Produsent

Non-sterile / No estéril / Non stérile / Nicht steril / Non sterile / Niet-steriel / Não estéril / Nesterilni / Niesterylne / Ej steril / Ikke-steril / Ikke-steril

Medical device / Producto sanitario / Dispositif médical / Medizinprodukt / Dispositivo medico / Medisch hulpmiddel / Dispositivo médico / Zdravotnický prostředek / Wyrób medyczny / Medicinteknisk produkt / Medicinsk udstyr / Medisinsk utstyr

Reference number / Número de catalogo / Numéro de catalogue / Katalognummer / Numero di catalogo / Catalogus nummer / Catálogo de número / Katalogové číslo / Numer katalogowy / Katalognummer / Katalognummer / Katalognummer

Unique Device Identifier / Identificador único del producto / Identifiant unique des dispositifs / Einmalige Produktkennung / Identificazione unica dei dispositivi / Unieke identificatiecode voor medische hulpmiddelen / Identificador único do dispositivo / Jedinečný identifikátor prostředku / Niepowtarzalny kod identyfikacyjny wyrobu / Unik produktidentifisering / Unik udstyrsidentifikation / Unik enhetsidentifikator

Authorised Representative in the European Union / Representante autorizado en la Unión Europea / Représentant autorisé dans l'Union européenne / Bevollmächtigter Vertreter in der Europäischen Union / Rappresentante autorizzato nell'Unione Europea / Gemachtigd vertegenwoordiger in de Europese Unie / Representante Autorizado na União Europeia / Zplnomocněný zástupce v Evropské unii / Autoryzowany przedstawiciel w Unii Europejskiej / Auktoriserad representant i Europeiska unionen / Autoriseret repræsentant i Den Europæiske Union / Autoriseret representant i Den europeiske union

INSTRUCTION FOR USE OF ULTRASONIC TRANSDUCER PROBE COVER

READ INSTRUCTION BEFORE USE

INTENDED USERS

- For healthcare professional use only.

INTENDED USE

- As a protective cover in avoiding contamination of ultrasonic transducer probes by bodily fluids. To facilitate cleaning and sanitization of the probes.

Do not reuse as there might be an increased risk of infection.

INDICATION FOR USE

- These ultrasonic transducer probe covers (commonly known as probe cover) are regulated as an accessory to the ultrasonic transducer probes of the Ultrasonic Imaging System, therefore it is necessary to read the probe reprocessing instructions and/or cross-referencing of probe manuals to ensure correct usage of the probe covers use with the probes.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.

DIRECTIONS FOR USE

- Check that the expiry date is current.
- Open the probe cover package by tearing gently and carefully.
- Make sure the rolled part of the probe cover to be unrolled is on the outside.
- Place the probe cover on the head of the ultrasonic transducer probe and unroll it gently all the way to the base.
- If lubrication is required, do not use oil based lubricants with this natural latex probe cover as oil-based lubricants may damage the probe cover. Use water-based lubricants.
- For single use only.

STORAGE

- Store in a cool, dry place away from direct sunlight.

DISPOSAL

- Please dispose of used probe cover hygienically and thoughtfully.

Do not flush into the toilet.

CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MIGHT CAUSE ALLERGIC REACTIONS INCLUDING ANAPHYLACTIC SHOCK.

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