 HENRY SCHEIN®

HELIX TEST KIT • INSTRUCTIONS FOR USE

Test Helix • INSTRUCCIONES DE USO

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


Helix Test • ISTRUZIONI D'USO

9008649 Helix Test Kit
9008650 Helix Test Refill
9008651 Helix Test Device

Conform EN 867-5, ISO 11140 - 1, Class 2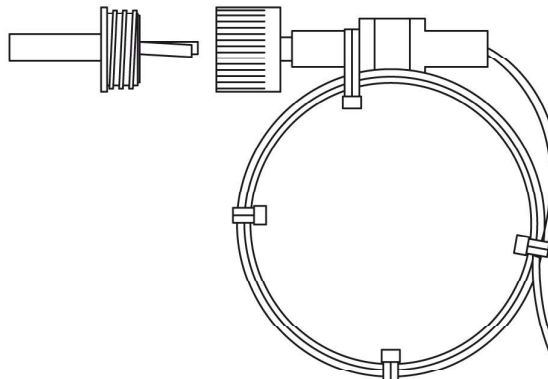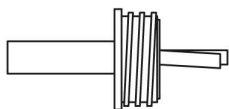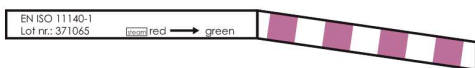

Instructions for use

NOTE: The Helix tube should not be used for more than 250 tests.

1. Check the device for visible damage and ensure that the device is dry. Also ensure that the device is not warm.
2. Open the tube by removing the capsule from the device.
3. Remove a strip from the sheet. Fold the indicator strip in half and ensure that the indicator faces inwards.
4. Place the strip into the capsule of the device. The folded side should be inside the chamber of the capsule and the open end faces to the open end of the capsule.
5. Ensure that the indicator does not move from the capsule.
6. Close the device.
7. Run a sterilization cycle as indicated on the helix test strip.
8. At the end of the sterilization cycle, remove the device from the autoclave. Be careful, the device may be hot.
9. Open the capsule from the device and remove the indicator. Check the indicator. The test is positive if the chemical indicator changes to the right color (red / pink → green).
10. The indicator strip has a self adhesive backing and can be applied to the sterilization records.

Distributed by (in US only)
 HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

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