SPHYGMOMANOMETER

• Instructions for use

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 HENRY SCHEIN INC.
(manufacturer)
135 DURYEA ROAD
Melville, NY 11747 USA

HENRY SCHEIN SERVICES GmbH

MONZASTRASSE 2A
D-63225 LANGEN, GERMANY
WWW.HENRYSCHEINBRAND.COM

Rev.2018/03

 HENRY SCHEIN INC.
(manufacturer)
135 DURYEA ROAD
Melville, NY 11747 USA

HENRY SCHEIN SERVICES GmbH

MONZASTRASSE 2A
D-63225 LANGEN, GERMANY
WWW.HENRYSCHEINBRAND.COM

Rev.2018/03

Users manual for SPHYGMOMANOMETER

INTRODUCTION:

This blood pressure monitor conforms to the non-invasive sphygmomanometers – Part 1: requirements and test methods for non-automated measurement types (EN ISO 81060-1).

All Calibration testing – at least every two years –, has to be carried out either by the manufacturers or the local authorised dealer in accordance with the relevant distributor directives.

IMPORTANT:

Keep this unit from being knocked or dropped, and protect it from dirt and moisture. Care has to be taken not to puncture the cuff with any sharp instruments (Scissors, Needles). Do not inflate to above 300 mmHg ! Measurement time should be ca. 2 minutes maximum. Between 2 measurements allow an interval of at least 2 minutes.

Please use only Henry Schein cuffs.

PLACING THE CUFF:

Place cuff in such a manner that the lower cuff edge finishes ca. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

TAKING THE BLOOD PRESSURE:

Inflate the cuff to above the systolic pressure. The air release speed can be adjusted by rotating the valve screw.

The World Health Organisation (WHO) recommends a deflation speed of 2 – 3 mmHg per second.

After the measurement is completed, fully open the air release valve to allow for the fast deflation of the cuff.

TECHNICAL DATA:

Measurement range: 0 – 300 mmHg

Accuracy of Pressure indication: ± 3 mmHg

Storing conditions: -30°C – 70°C , 15–85% rel. humidity

Operating conditions: 20°C – 70°C , 20–85% rel. humidity

CLEANING OF UNIT AND CUFF:

- To clean the unit only use a dry, soft cloth.
- To clean the cuff – remove the rubber bag. The exterior of the cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

DISINFECTION:

We recommend non-alcoholic disinfection wipes or liquid to disinfect the device.

CUSTOMER SERVICE:

For repair work, please send the instrument carefully packed to you local Henry Schein Office.

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