Maxima® by  HENRY SCHEIN®

Maxima Non-Alcoholic Surface Disinfection Concentrate

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LOT

CE 0473 250 ml e

Microbiological Effectiveness/Contact Time: Bacteria (incl. MRSA) according to VAH/DGHM, EN 1040, EN 1276, EN 13697 (high bio load): 1min. Candida albicans according to VAH/DGHM, EN 1275, EN 1650, EN 13697 (high bio load): 1min. membrane covered viruses (i.e. HBV, HIV, HCV, H1N1, H5N1) according to BGA (RKI)/DWV (with bio load): 1min. (with bio load). Rota-viruses according to BGVV/DWV: 1min. Noro-viruses according to EN 14476 (clean): 30min. **100g contains:** 0,24g Dialkyldimethylammoniumchlorid 0,48g Alkyl(Aryl)dimethylammoniumchlorid

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


Maxima Non-Alcoholic Disinfection Wipes Container

- Alcohol free wet wipes for rapid surface disinfection of non-invasive medical device.
- Toallitas húmedas sin alcohol para la desinfección rápida de productos no invasivos sanitarios.
- Désinfection rapide des petites surfaces, des dispositifs médicaux non invasifs et des fauteuils dentaires.
- Alkoholfreie Feuchttücher zur schnellen Desinfektion von nichtinvasiven Medizinprodukten.
- Fazzolettini umidi privi di alcol per una rapida disinfezione dei dispositivi medici non invasivi.
- Alkoholvrije vochtige doekjes voor snelwerkende desinfectie van niet-invasieve medische hulpmiddelen.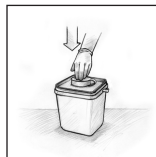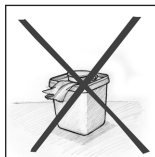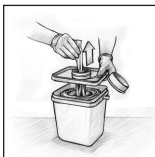

DISTRIBUTED BY:

DISTRIBUIDO POR: VERTEBRO DURCH:
DISTRIBUÏE PAR: HENRY SCHEIN INC.
MELVILLE, NY 11747 USA

EC REP
HENRY SCHEIN U.K. HOLDINGS LTD.
GILLINGHAM ME8 0SB U.K.
WWW.HENRYSCHEINBRAND.COM

75 wipes
250 ml e non-alcoholic
Surface disinfection concentrate

Indications for use: Surfaces should be thoroughly wiped with the disinfectant tissue and left to dry. For larger areas of sensitive material such as artificial leather, try on a smaller area first to test compatibility, to avoid streaks from appearing a regular wiping with a wet (water) disposable cloth or tissue is recommended. **Microbiological Effectiveness:** Bactericidal (incl. MRSA) according to EN 1040, EN 1276, EN 13697; Levurocidal (Candida albicans) according to VAH/ DGHM, EN 1275, EN 1650, EN 13697; "restricted viruses" (effective against membrane covered viruses, i.e. HBV, HIV, HCV, H1N1, H5N1) and effective against Rota viruses. **Contact Time:** Bacteria (incl. MRSA) according to VAH/DGHM, EN 1040, EN 1276, EN 13697 (high bio load): 1 min. Candida albicans according to VAH/DGHM, EN 1275, EN 1650, EN 13697 (high bio load): 1 min. Membrane covered viruses (i.e. HBV, HIV, HCV, H1N1, H5N1) according to BGA(RKI)/DVV (with bio load): 1 min. Rota-Viruses according to BGW/DVV: 1 min **100g saturated fluid contains:** 0,24g Dialkyldimethylammoniumchloride 0,48g Alkyl(Aryl)dimethylammoniumchloride Packing material should be completely emptied of contents before disposal. **Content:** 75 Tissues, 1150g saturated fluid

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