ULTRASOUND GEL

- CONTACT AGENT FOR ULTRASOUND DIAGNOSTICS AND THERAPY. APPLY THE ULTRASOUND GEL EVENLY • FOR EXTERNAL USE ONLY

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EN - Instructions for use Ultrasound gel

The contact gel is a non-sterile, water-soluble and non-staining gel that is intended exclusively for external use and can be used with commercially available ultrasound devices.

Areas of application

This contact agent for ultrasound therapy and diagnostics can be used on intact, undamaged skin in order to achieve an loss-free a transition as possible from the transducer to the body. The contact gel is intended for use by healthcare professionals. Based on the available evidence, there is no exclusion of patient groups.

Application

Spread the required amount of gel on the skin surface of the body area to be examined and/or on the ultrasound probe. Carrying out the ultrasound examination/therapy. Then remove the gel from the ultrasound probe and the patient/body area with a soft cloth. Shelf-life: 6 months after opening.

Contraindication

The gel is intended for external use only and should not be used for the following applications:

- Invasive procedures in which a device is inserted through the tissue.
- Any procedures on or near damaged skin.

Ingredients: Aqua, glycerin, carbomer, sodium hydroxide, propylene glycol, benzyl alcohol, methylchlorisothiazolinone, methylisothiazolinone, citric acid

INSTRUCTIONS FOR SAFE USE/precautions

In rare cases, contact gels may cause temporary skin irritation to sensitive individuals. Allergies should be clarified with the patient in advance.

To avoid contamination, the tips of the containers/tubes must not come into contact with patients, staff or instruments.

Dispose of the product if the packaging is damaged or there are signs of contamination.

Wipe the turret cap after use. For dispensers and refill containers, follow the enclosed instructions for use or your facility's protocol if the instructions differ from the facility's protocol.

Reporting of incidents

You should report any serious incident that occurs in connection with the ultrasound gel to the manufacturer

er (e-mail: qm@cundv.de) and your competent authority.

The ultrasound gel is available in the following container sizes.

250 g dispenser bottle / 500 g dispenser bottle / 1000 g dispenser bottle / 5 kg Cubitainer / 10 kg Cubitainer / 5 kg canister / 10 kg canister

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C+V Pharma Depot GmbH