# HOT / COLD PACK

ES	BOLSA DE FRÍO Y CALOR
FR	PACK CHAUD ET FROID
DE	HEISS- UND KALTKOMPRESSE
IT	BUSTA PER CRIOTERAPIA E TERMOTERAPIA
NL	HOT & COLD PACK
PT	COMPRESSA QUENTE E FRIA
CS	HŘEJIVÝ A CHLADIVÝ SÁČEK
PL	GORAŻE I ZIMNE OKŁADY
DA	VARMT OG KOLDT KOMPRES
SV	VÄRME- OCH KYLPÅSE
NO	VARME OG KALDE OMSLAG

## Intended use

### Product Structure

The products are made up by content and package. The main component are sodium chloride, CMC, pure water.

### Intended use

Used for decrease temperature by physiotherapy, improve wrench and light scald. Warm and Thermal treatment for adult. Apply for relieving pain of lower back, neck, somatoform and chronic pain, assisting the post exercise recovery and the recovery of arthritis, strain.

### Intended users

Adult

### Contraindications

Not apply for injuries with inflammation.  
Not apply directly for skin, or injured skin with bleeding.

### Cautions and Warnings

- The product only for external use, if inner liquid touch eyes or skin when the packing broken, just washing by clean water. If the pack is punctured or opened and the solution is accidentally swallowed, drink plenty of water.
- Don't puncture the pack with sharp object, never pressed by heavy object deliberately in case of damaging. Do not strike or stand on pack.
- Eating forbidden.
- Do not use in injuries, sprain when inflammatory exudation still existing in 24 hours. Do not use on sensory impaired patients.
- If acute abdomen ache is not confirmed; triangle area on face infected; any viscera bleeding or parenchyma contusion etc, avoiding hot therapy.
- Keep out of the reach of children. Always use under adult supervision.
- If you are affected by circulatory problems seek medical advice before use.
- Cease use if pack causes any discomfort.
- Consult your doctor if you have any concerns about your health.

### How to use

#### Cold therapy:

Put the product in the icebox or freezing box (minus 15 centigrade) of refrigerator for more than 2 hours.

#### Hot therapy:

Hot Water Method:

Immerse pack in boiling water for approx 30 minutes, remove pack from water and always inser the hot/cold pack into towel cover provided.

It is recommended that the maximum temperature not exceed 60°C, hot or cold compress time for 15~20 minutes. Repeat every 2 hours as needed.

Reprocessing instructions: After each use, dry off and store in the refrigerator or in a cool place. It can be reused at least 50 times.

### Shelf life

3 years.

### Storage conditions

Store the product in shadow and dry (the finished products should be stored in the well-ventilation place without corrosive air where the comparative humidity is not more than 80% avoiding irradiation by ultraviolet). Without special requirement, non-dangerous good, can be normal transportation by air, sea and road.

### The disposal requirement of the device

All waste must be handled in accordance with local, state and federal regulations

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**HENRY SCHEIN SERVICES GmbH**